


# SARS-CoV-2 Antigen Test Kit (Colloidal Gold)

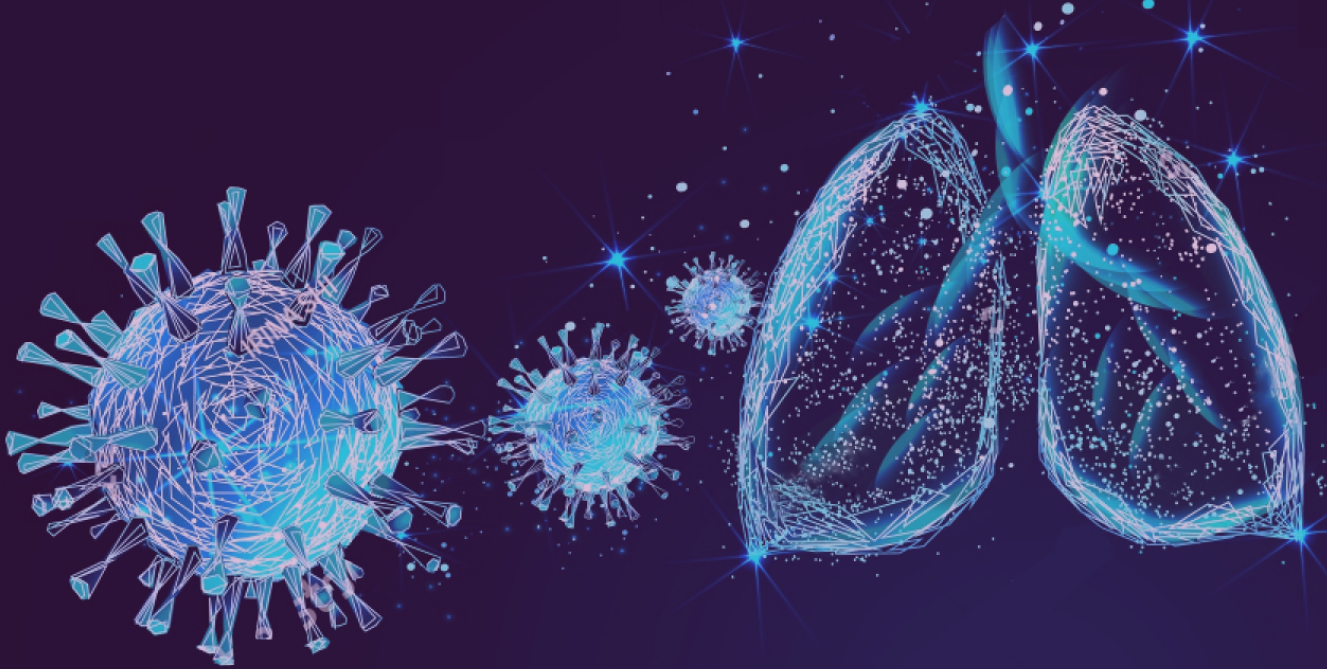

The Genrui SARS-CoV-2 Antigen Test Kit (Colloidal Gold) is a rapid lateral flow immunoassay to qualitatively detect the specific antigens to SARS-CoV-2 in human respiratory tract, help to screen or identify acute or early infection of novel coronavirus.



# SARS-CoV-2 Antigen Test Kit (Colloidal Gold)

## Specifications

Sample: oropharyngeal swab , nasopharyngeal swab


Testing time: 15-20min

Storage condition: 2-30°C


## Key features

- **SIMPLE** - Can be performed by healthcare workers at anytime, anywhere without additional equipment
- **FAST** - Quickly identify COVID-19 to flatten the curve
- **RELIABLE** – Good sensitivity and perfect specificity compared to PCR test results

## Test procedure

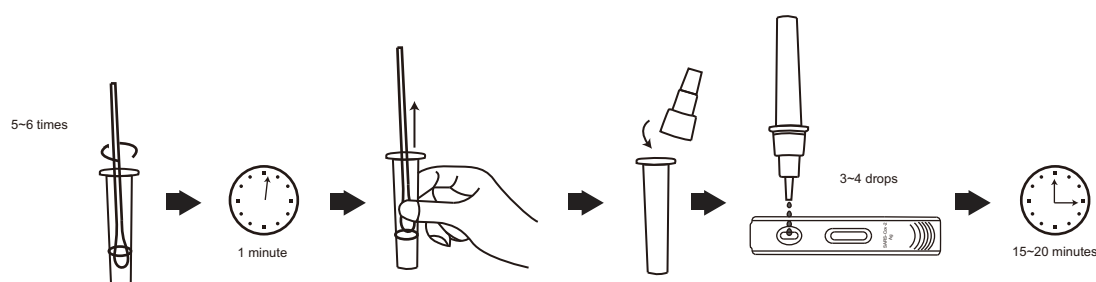

## Test results

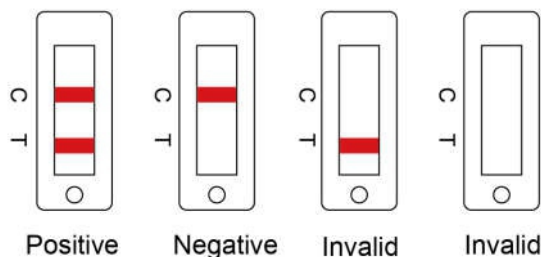

## What makes it reliable

386 clinical samples confirmed by PCR test were compared by Genrui SARS-CoV-2 Antigen Test Kit (Colloidal Gold) . The results are shown as below.

		PCR		Subtotal
		Pos	Neg	
SARS-CoV-2 Antigen Test Kit (Colloidal Gold)	Pos	172	2	174
	Neg	9	203	212
Subtotal		181	205	386

Positive Percent Agreement: 95.03% (95%CI: 90.77%~97.70%)

Negative Percent Agreement: 99.02% (95%CI: 96.52%~99.88%)

Overall Percent Agreement: 97.15% (95%CI: 94.96%~98.57%)

## Order Information

Product No.	Product Name	Package size	Certification
P04.03.109387-00	SARS-CoV-2 Antigen	25T	CE